





THE BUCCAL SWAB SPECIMEN COLLECTION

1

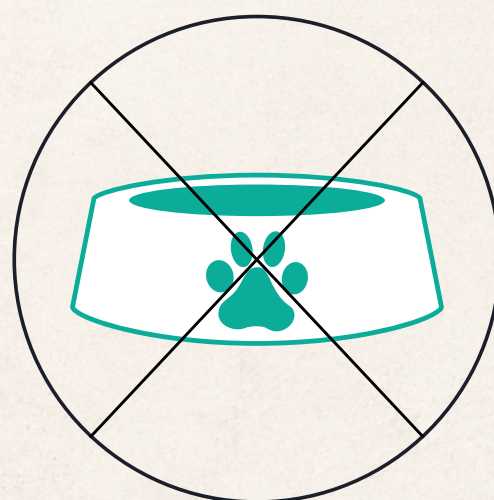


Collection must be carried out on an empty stomach or 2 hours after meal.

2

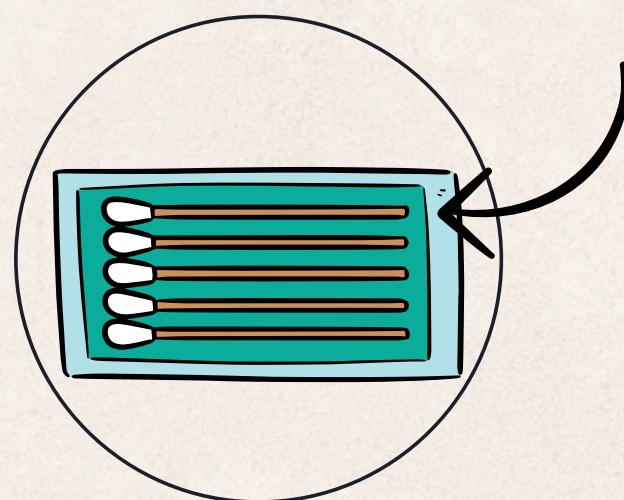


Carefully peel apart the sleeve containing sterile buccal swabs.

3

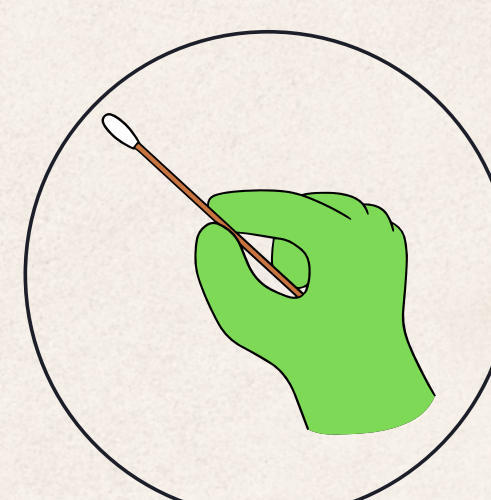


Remove the buccal swab carefully holding the swab by the stick.

4

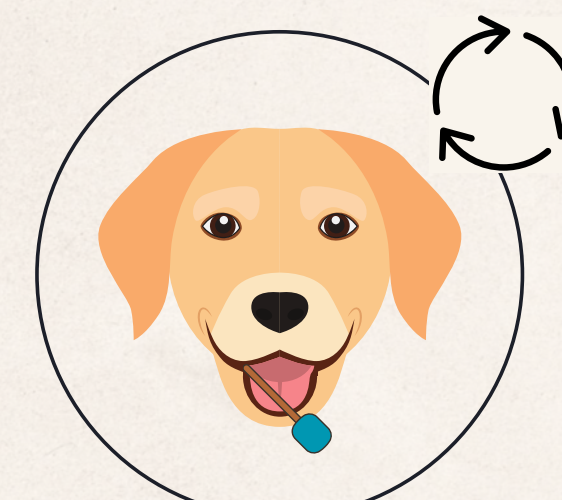


Gently rotate the swab on the inside of the animal's right cheek for 15 seconds. Repeat for remaining swab, this time collecting from the left cheek.

5

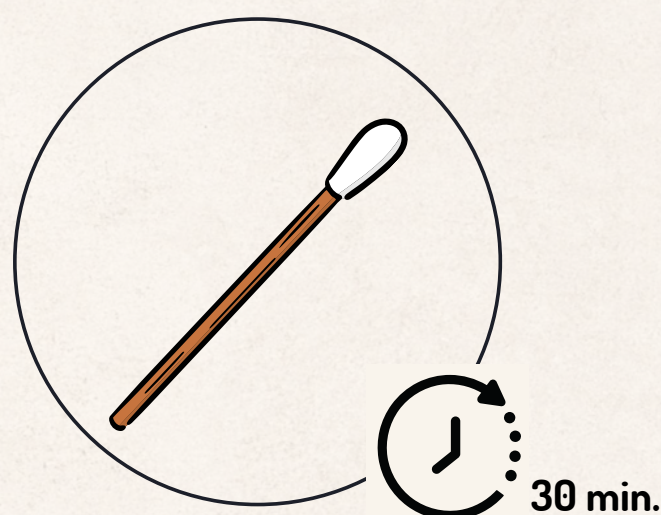

Allow the buccal swabs to air dry for approximately 30 minutes. Avoid any contact of the cotton end with the surroundings.

6

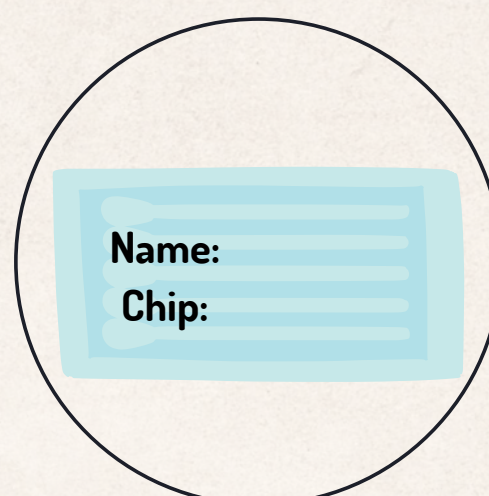

Place dry buccal swabs into their original packaging. Label the packaging with the animal's name and/or chip number.

7



Print and fill out the order form.

8



Send the labeled sample together with the order form to the laboratory.